

Products affected


- DCS 2.x.x (version 2.1.0) and Cardio Connex™ 1.x.x (version 1.0.0 to 1.1.1)

Notes

- This instruction is applicable for systems currently configured with the logon authentication method set to **Active Directory** and with **Single Sign On (SSO)** turned on.
- This process will briefly interrupt service. Perform these steps at an appropriate time.
- After turning off SSO, users will then need to enter their account name and password when logging into the system.

Steps to turn off SSO

1. Log into Windows as an administrator.
2. In the Start menu go to **Hillrom** and then select **Connex Cardio Configuration Tool**.
3. When prompted that services will be stopped, click **OK**.
4. The **Modality Manager Configuration Utility** window will be displayed.

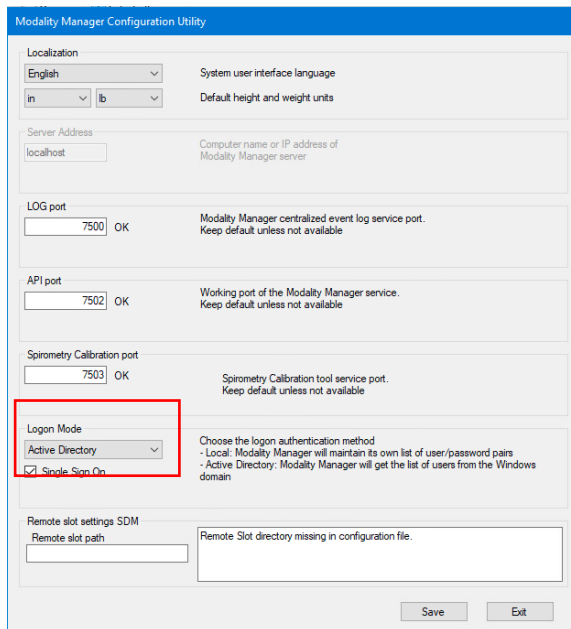

The screenshot shows the 'Modality Manager Configuration Utility' window. The 'Logon Mode' section is highlighted with a red box. It contains a dropdown menu with 'Active Directory' selected and a checked checkbox for 'Single Sign On'. Below this, there is a text box for 'Remote slot path' and a message: 'Remote Slot directory missing in configuration file.' The window also has 'Save' and 'Exit' buttons at the bottom.

5. Make sure **Single Sign On** is off.
6. Click **Save** then **Exit**.