
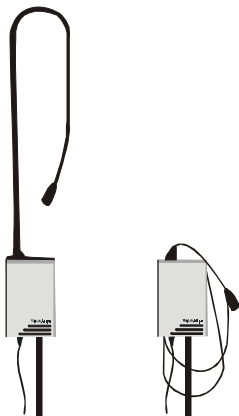


EXAM LIGHT III 35W HALOGEN LAMP REPLACEMENT INSTRUCTIONS for Models:

No. 48700**No. 48780****No. 48760****No. 48740****No. 48722****No. 48782****No. 48762****No. 48742****No. 48724****No. 48784****No. 48764****No. 48744****No. 48726****No. 48786****No. 48766****No. 48746****No. 48728****No. 48788****No. 48768****No. 48748**

**WARNING: LAMPS OPERATE AT A HIGH
TEMPERATURE. DO NOT ATTEMPT TO REPLACE LAMP
WHILE HOT. WAIT UNTIL LAMP IS COOL.**


1. Turn the unit off and unplug the power cord from both the wall outlet and the Light Box.
2. Remove the unit from the patient area.
3. Using a screwdriver, loosen the single panel fastener that protrudes from the bottom of the unit. As the screw is loosened, there will be a noticeable “drop” as the front cover is released. **(A)**
4. Slightly lift the bottom of the front cover toward you and then slide the cover up to remove it. **(B)**
5. For initial lamp replacement only: Grasp wire clip at top of lamp. Rotate clip toward the base of the unit until it pops out. **(C)** Discard clip. It was for shipping purposes only.
6. Grasp the lamp connector and pull the lamp straight out of its holder. **(D)**
7. Pull the lamp from its connector and replace it with Welch Allyn Replacement Lamp #04200. **(E)**
8. Push the lamp back into the holder.
9. Attach the front cover by sliding the tabs at the top of the cover into the slots at the top of the back housing and then tightening the screw at the bottom of the unit.
10. Return the unit to the patient area and plug power cord back into bottom of unit and wall outlet.

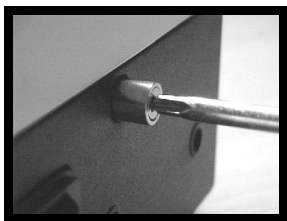


(A)



(B)

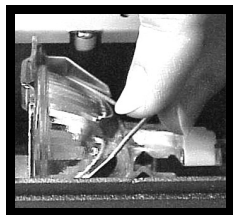

(C)

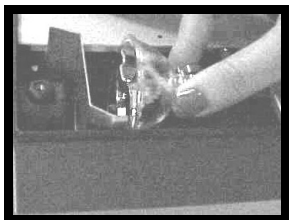

(D)



(E)